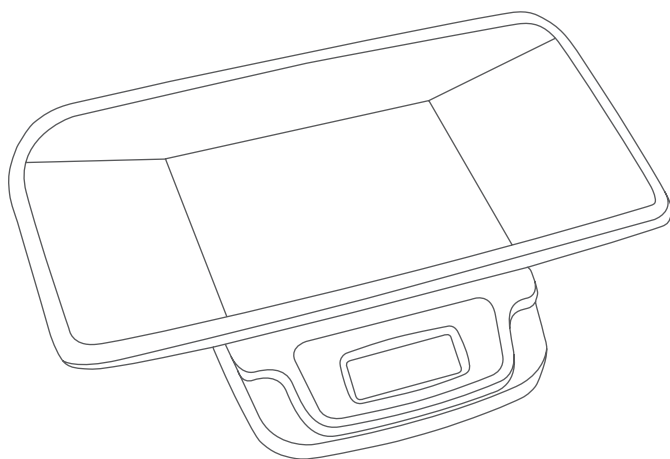
GRAM

SERIES / SERIE / SÉRIE

WHITE MEL

Baby scale / Pesabebés / Pèse-bébés

EN

ES

FR

CE

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INDEX**English**

Technical specifications _____	4
Keyboard description _____	4
Operation _____	5
Error messages _____	6
Cautions _____	6
Guarantee _____	7

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
ENGLISH

TECHNICAL SPECIFICATIONS

- **Work temperature:** 10-40°C
- **Environmental humidity:** <80%
- **Maximal capacity:** 20 kg
- **Graduation:** 5 g
- **Power supply:** 4 AA (R6) batteries
- **Low power message:** L o
- **Error of indication:** Err
- **Unit weight:** kg or lb
- Auto turn-off function.

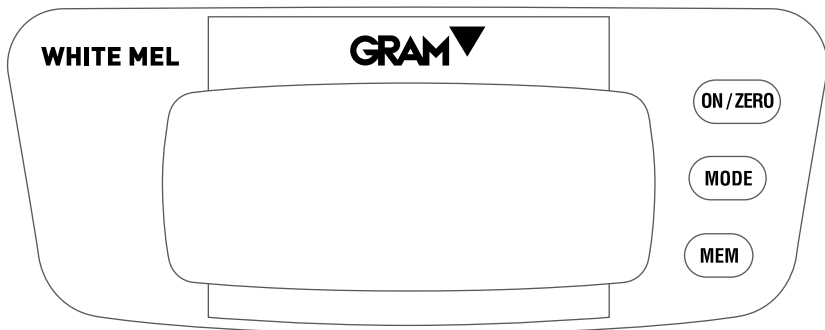KEYBOARD DESCRIPTION

ON / TARE

Press this key to turn on or turn off the balance.

Use it also to zero the display reading.

To turn off the balance, keep it pressed for 3 seconds.

MODE

Press this key to select the available unit weight (kg. or Lb).

Once selected the unit weight, his symbol will appear on right side of display.

MEM

This key allows to save and recall the baby weight.

Place the baby on the weighing pan, when the weight is stabilized, press the **MEM** key.

When display is at zero **"0.000"**, press again this key to recall the baby weight saved on memory.

OPERATION

- Place the batteries.
- Press the **ON/TARE** key to turn on the balance, the display will show **"0.000"**.
- Carefully place the baby on the weighing pan, the display will show his weight. If the baby is still on the plattform, the value will remain illuminated for 10 seconds, after this time, the balance will automatically turn off.
- When the baby is removed from weighing pan, the display will again show **"0.000"**.
- The balance will turn off after 10 seconds of not use.
- To recall the weight saved in memory, with reading display at zero, press the **MEM** key.

ERROR MESSAGES

L o o.

Means that batteries are dead and they must be replaced immediately with new ones.

E r r.

If this message appears, turn off the balance and turn on again without any object on the pan.

CAUTIONS

- For comfort and safety of the baby, place the balance on a smooth and solid surface.
- When displays shows “**L o o**”, change immediately the batteries.
- Keep the balance clean to ensure a good operation.
- If the balance is not used for a long period of time, remove the batteries.
- Place the baby in the center of weighing pan to obtain best results.

GUARANTEE

The scale is warranted against defects in material and workmanship for a period of 1 year from the date of delivery.

During the warranty period, **GRAM PRECISION SL** will repair any product that proves to be defective.

Limitation of warranty

This warranty does not apply to defects resulting from misuse of any product or part. This warranty also does not apply to damage from overload or problems arising from failure to follow instructions (Especially concerning the **MAINTENANCE TIPS**).

The warranty does not cover shipping costs necessary to repair the scale. Therefore, shipping charges to and from the repair location must be paid by the owner.

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